Everyclean is the best cleaner optimized for the sintering furnace with MoSi₂ heating elements. By eliminating remaining coloring liquid in the chamber or heating elements, you can get more transparent and ideal crown shade. Everyclean consists of 12 bottles (100g) with different mark for each month so that cleaning can be done monthly bases. If you clean sintering furnace with Everyclean once in a month, you can minimize contaminant in the chamber or heating elements. For example, use Everyclean with mark 'May' when you do cleaning in May. Then, you will never forget monthly cleaning. After cleaning, the orange color Everyclean will be turned into white color in normal condition of sintering furnace. With Everyclean the best zirconia crown will always be with you.

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12 bottles of Everyclean have different mark by month. Choose Everyclean bottle marked with 'May' when you do cleaning in May. This 12 months mark will help you keep monthly cleaning without

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Put all the Everyclean(100g) in sintering tray after removing sintering bead and sintering powder.

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Regarding sintering schedule, pick up Zirconia sintering schedule and then start operation.

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After cleaning, you can find out the orange color Everyclean has been turned into white color. This change means that cleaning has been done well. And also, Everyclean which absorbed contaminant may have partially yellow or green color. If you make monthly cleaning with Everyclean, you can get rid of this phenomenon.

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Put date of cleaning on Everyclean tag enclosed in box and then hang/keep it on handle of sintering furnace, which enables you to do monthly cleaning